


# TV Ears Home Theater Installation Instructions

1. Place the **Power Stackers** on top of each other, with the lights facing out into the room.

**Note:** Be sure the TV Ears Audio Cord is plugged into the top Power Stacker in "Audio" port.



Audio Cord in top Power Stacker "Audio" port

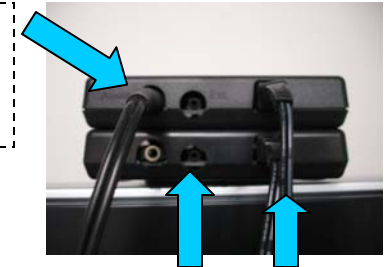

"Extension" ports

12v port

2. Plug the **AC Adapters** into outlets near your TV. Plug the other end into the "12v" port on the back of the **Power Stackers**. The "12v" port is the socket on the right of the **Power Stacker**.



12v port w/ AC Adapter plugged in

3. Plug the **black** end of the short **Patch Cord** into the **Power Stacker** on the bottom. Plug the end of the **Patch Cord** with the **Red** tape into the top **Power Stacker**.

**Note:** When plugging in this cord make sure that you feel it "click" into place. Make sure that all of the **Infrared Lights** are glowing on both **Power Stackers**.



Black end of Patch Cord plugged into bottom Power Stacker

Red tape end of Patch Cord plugged into top Power Stacker

Patch Cord

