

T - Coil Module



T-Coil into
Input jack



1. Plug the T-Coil Module into the input jack under the 'tone' dial on your TV Ears headset.

2. The T-Coil Module will receive audio from looped facilities and telephones and transmit it into the headset.



Output jack

Input jack

Neck Loop



Neck Loop into
Output jack

1. Plug the neck loop into the output jack under the 'volume' dial on your TV Ears headset.

2. Place the neck loop around your neck along with your TV Ears headset.

3. The neck loop will interact with hearing aids equipped with a T-Coil.

